



Datt Mediproducts Private Limited

VELNEZ[®]

The Optimal Healing Solution



**The painless solution to manage
surgical procedures**

VELNEZ®

The Optimal Healing Solution

VelNez® is a biodegradable composite that degrades slowly on application and is intended as an ear dressing. The pain associated with the traditional ear plug removal is completely eliminated since such a procedure is not necessary with VelNez®. It also reduces fibrosis and at the same time promotes healing and reduces the clotting time. During ear procedures, it can be applied to control compression of tissue, middle ear structure and prevents adhesion.

INTENDED USE

VelNez® is intended to be used as a packing dressing in the middle and outer ear cavity after surgery. It acts as a space occupying dressing, preventing adhesion by separating the compromised surfaces. It also helps in healing and achieving haemostasis.

INSTRUCTIONS FOR USE

Step 1: Removal from Package – Open the blister pack by removing the paper film and take out VelNez®.

Step 2: Pre-insertion - If required cut the VelNez® using sterile apparatus.

Step 3: Insertion - Using sterile apparatus, place VelNez® inside the ear cavity at the site of application.

Step 4: Confirm - Confirm that the device is placed at the intended position. VelNez® will fill itself by absorbing the surrounding fluids and blood.



CE
2460

Note:

- The equipment/ instruments required for VelNez insertion can be used as per clinician discretion.
- Upon degradation, VelNez comes out from the middle ear to outer ear cavity in a discharge form. The residuals in the outer ear cavity can be cleaned during post-op care by the healthcare provider or patients.

CONSTITUENTS

Gelatin (BSE/TSE agent free) Chitosan, Polyvinyl Alcohol, Psyllium Husk.

WARNING & PRECAUTIONS

- Do not re-sterilize.
- Do not re-use.
- Use the device prior to the "Use by (M)" specified on the package.
- To be used by Registered Medical Practitioner or a hospital.
- Do not use if package is damaged.
- Discontinue the use if the application area shows signs of infection, irritation, maceration (whitening of surrounding skin) or itching & consult a healthcare professional.

RISK OF REUSE

VelNez® is supplied sterile for single use only. Do not re-sterilize as this may change product characteristics and lead to failure of intended use of the product. Damaged, unused, opened packages of VelNez® should be discarded as per disposal instructions. Sterility of the product is not guaranteed when the packaging is damaged or unused portion of the device is left in the pack.

INDICATIONS

As ear pack for hemorrhage control or/and prevention of adhesion between the compromised surfaces after an ear surgery, traumatic bleeding, Tympanoplasty, Tympanostomy or Myringoplasty and other inner and middle ear surgeries.

CONTRAINDICATIONS

- Should not be used for the patients with coagulation disorders.

Note: Do not use VelNez® if patient is allergic (Hypersensitive) to it's constituents.

PATIENT POPULATION

Patients with age group from 18-60 years of age.

INTENDED USER

Surgeon, ENT specialist.

HOW SUPPLIED

Velnez is supplied as a sterile, single-use product in one pack.

STORAGE CONDITIONS

VelNez® should be stored at 4°-30°C. Keep away from direct sunlight. Store in a dry place.

SHELF LIFE

3 years from the date of Mfg.

METHOD OF STERILIZATION

Gamma Irradiation.

AVAILABLE SIZES

SHAPE	SIZE	REF
Square	4cm × 4cm	140404ER
Square	2cm × 2cm	140202ER
Rectangular	2cm × 1cm	140201ER
Cylindrical	20mm × 10mm	142010ER
Cylindrical	20mm × 5mm	142005ER
Cylindrical	20mm × 3mm	142003ER

PATENTED

For more information write to Marketing Department



Datt Mediprodukt Pvt. Limited
(An ISO 9001 & 13485 Certified Company)

Admin Office: 56, Community Centre, East of Kailash, New Delhi-110065
Works: 52-54 & 63-64, Roz-ka-Meo Industrial Area,
Nuh, Distt. Mewat-122 103, Haryana, India
E-mail: advancedevice@dattmedi.com | Website: www.dattmedi.com
Customer Care Toll Free No.: 1800-1030-707

ECIREP

mdi Europa GmbH Langenhagener Str. 71
D-30855 Hannover-Langenhagen, Germany Phone: +49-511-3908 9530
Internet: www.mdi-europa.com Trade Register Hanover HRB 58423